



Overbed Table

GM-HMP-S209

Instruction Manual

Read this manual carefully before using this product

Goldfern*Mobility*[™]

www.goldfernmobility.co.nz

Welcome!

Thank you for your purchase of our Overbed Table.

Please find the following instructions for use and product specifications. Please read this owner's manual carefully before using your Overbed Table. This manual contains important safety information and valuable tips for proper use and care. These operating instructions contain all of the important information for adjustment and operation.

Contents

Welcome!	01
Product Details & Safety Guidelines	02
User Instructions	03
Cleaning, Maintenance & Measurements	04

Product Details

Scope of Delivery

- 1x TwinTop table top
- 1x Base frame made of steel tubing
- 3x Star screw
- 1x Owner's manual



Assembly

The Overbed Table is delivered already partially assembled. You only need to connect the table top **(1)** to the base frame **(2)** and secure with the star screw.

Warranty

The warranty period for the product supplied by Goldfern Mobility is 12 months from the date of purchase. Parts subject to wear* are not covered by the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error. We reserve the right of error and modifications on the basis of technical improvements and changes in the design.

***Parts subject to wear (among others):** Casters, breaks

Safety Guidelines

The symbols below are used throughout this owner's manual to identify warnings and important information. It is very important for you to read them and understand them completely.



WARNING! Indicates a hazardous situation that could lead to serious injury or death if it is not avoided.



IMPORTANT! Indicates a hazardous situation that could result in damage to property or table if it is not avoided.



CAUTION! Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

User Instructions

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user. This product must be checked by the specialist dealer, cleaned, and restored to optimum condition. Always follow the manufacturer's instructions during preparation; these are available upon request.

The service and side table can be used in combination with a patient or hospital bed. It is intended for users as well as patients and persons in need of care as a valuable aid, for example, during body care, when eating and drinking in bed, or also during therapeutic measures.

! CAUTION: Persons with significantly limited cognitive abilities should use the service and side table only under supervision.

Adjusting the Table

To change the tilt angle, release the adjustment lever with one hand and adjust the tilt angle with your other hand. The tilt angle can be adjusted in 7 different positions.

To adjust the table height, loosen the star screw by rotating it counterclockwise. Pull the fixation lever out and adjust the table top to the desired height. Now allow the fixation lever to click into place in the set position. In addition, secure this position by tightening the star screw clockwise. The height of the table top can be adjusted in 10 positions between 71cm and 93cm.

Securing the Standing Position

Secure the table by using your foot to secure the breaking casters by pressing down on the brake.

! IMPORTANT! **ALWAYS** secure the breaking casters when the table is not being moved.

Caution

- Before assembly, check the delivery contents, in particular the assembly materials
- **ALWAYS** comply with the 10kg maximum central load of the service and side table
- **DO NOT** use the service and side table to assist from getting up from a sitting or lying position
- **DO NOT** lean or support yourself on the service and side table
- When stationary, **ALWAYS** secure the braking casters
- Use by children without adult supervision is prohibited - risk of pinching and tripping
- Regularly check the stability and sturdiness of the product and, where applicable, the secure fit of the screws

Cleaning & Maintenance

Cleaning the Table


The service and side table can be cleaned and disinfected manually. To do so, use a household cleaner and a damp cloth.

The service and side table do not require any special maintenance or harsh cleaning products/tools.


Repairs & Maintenance

Repairs as well as some adjustments require special technical training and must therefore be made by a professional.

Measurements

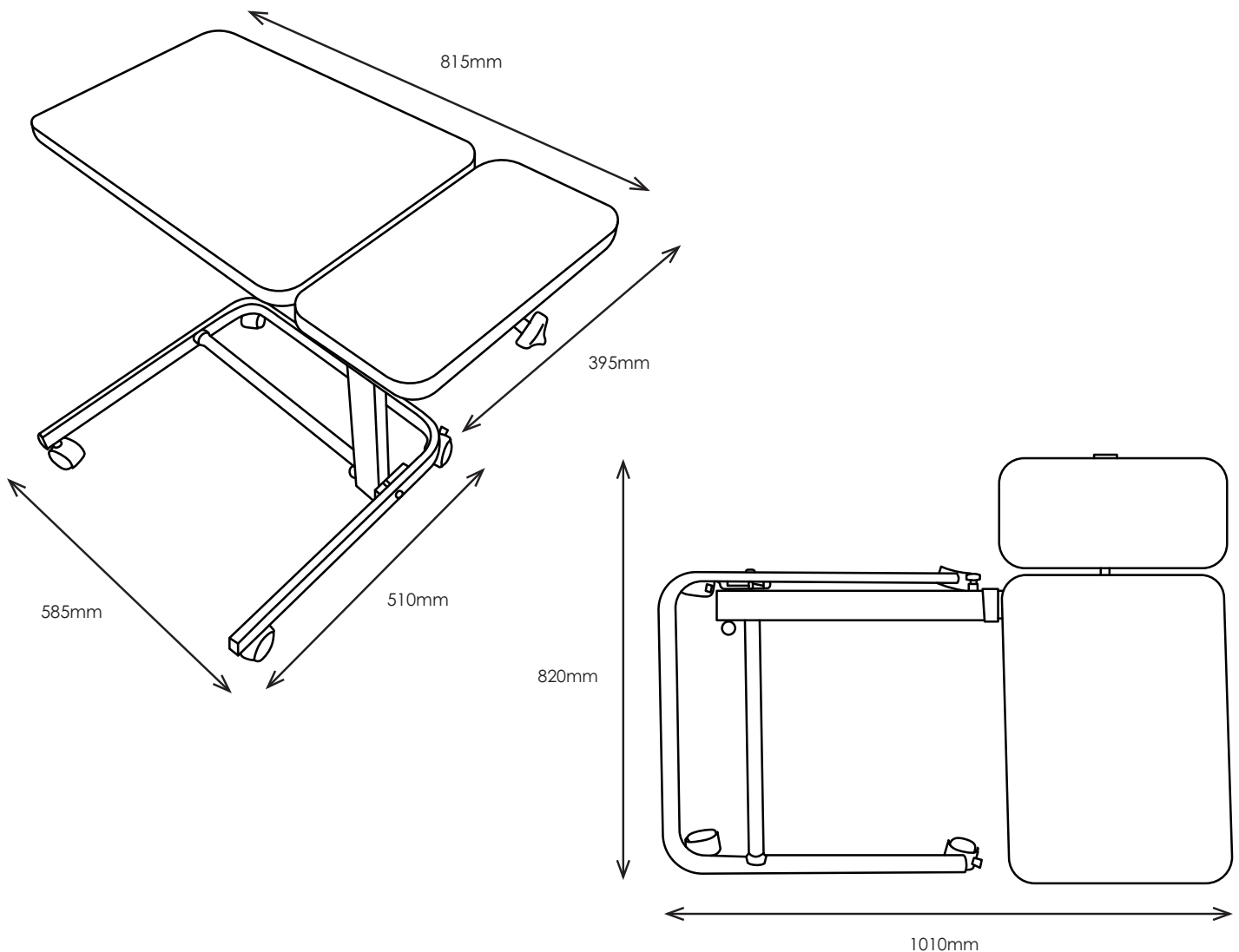

Goldfern*Mobility*[™]

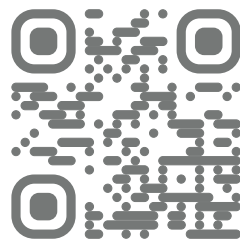

Goldfern Mobility
208 John F Kennedy Drive
Palmerston North
New Zealand

0800 100 971
www.goldfernmobility.co.nz