GoldfernMobility

ASSEMBLY INSTRUCTIONS OVER BED TABLE HMP-S207

1

Remove the contents of the box and lay out the pieces (Figure 1 and Figure 2).



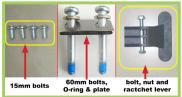


Figure 2

2.

Fit the base to the bottom of the mast with the split O ring/washer between the plate and the bolt head (60mm), tighten with Allen Wrench (Figure 3).



Figure :

3.

Assembling the table top to the mast (Figure 4).

- 3.1 Unscrew the bolt, nut and ratchet lever.
- **3.2** Slide the table into mast (Figure 5).
- **3.3** Assemble table to mast by using 4 bolts (15mm) with Allen Wrench (Figure 6).
- **3.4** Re-assemble bolt, nut and ratchet lever to the mast (Figure 7).





Figure 7

4

Remove the black locking screw of the mast and replace with sliver color screw (Figure 8). Operating Instructions. Please ensure that the hand nut (figure 11) is loose before altering table top height.





Figure 8

5

(Figure 9) shows the table fully assembled. The table height can be adjusted by raising the ratchet lever and lifting the table top at the same time, when desired height is reached release ratchet lever (Figure 10). To lower table top raise the ratchet lever and push down on the table top.





Figure

6

To allow free movement of Table top up or down loosen hand nut (Figure 11). When thetable top is at a suitable height, tighten hand nut (Figure 11).



Figure 11

CAUTION!

The over bed table is not designed to support heavy weights. Users must not lean on the over bed table for support when getting up or sitting down. Safe Working Load 10kg