

Locking SHARK

SHARK's lock function is disabled (off) as supplied from the factory. However this programmable feature can be enabled and used to prevent unauthorized people from turning SHARK on.

TO LOCK SHARK



While the power is ON, press and hold the Power button for 4 seconds. The display will turn off immediately. After 4 seconds all LED's will flash briefly and the horn will sound a short beep. The powerchair will then turn off.

TO UNLOCK SHARK



While SHARK is locked, press the Power button to turn SHARK on. All LED's will flash briefly. The Battery Gauge LED's will then perform a slow right-to-left chase.



Press the Horn button twice before the countdown is completed (approximately 10 seconds).

The current state-of-charge will then be displayed and SHARK may be operated normally.

If the user does not press the Horn button twice before the countdown is complete, the Horn will sound a short beep and SHARK will turn itself off. The unlock sequence must be completed successfully before SHARK will drive again.