

Goldfern*Mobility*[™]

www.goldfernmobility.co.nz



Dual Motor Electric Lift Chair

Installation & User Manual

Standard **GM-HYE8063** | Heavy Duty **GM-HYE8063HD**

Read manual carefully before use.

Safety Guidelines

- The chair must be assembled correctly according to the Assembly instruction enclosed.
- The manufacturers of the chair, their agents and distributors are not liable for any damages caused by improper assembly of the chair.
- The chair must be assembled by using all the parts provided.
- The chair must be placed on a flat floor level.
- The chair must be in a dry room.
- Do not expose this chair to direct sunlight, stoves, heaters or any other heating appliances.
- Do not use the chair outdoors.
- When the chair is in storage, disconnect the power plug from the main supply to avoid overheating.
- Do not turn off chair over night.
- Do not use this chair in combination with other therapeutic or beauty equipment such as massage mat, slimming machine or electric blanket.
- Do not allow children or pets to play on or around the chair. This chair is not for children.
- Do not place any heavy object including children on the armrest, footrest, or backrest.
- Do not jump or play on the chair.
- Do not rock the chair, it is not a rocking chair.
- Do not sleep on the chair as opposed to a bed.
- Do not use the chair when your body is wet or damp.
- Do not connect or disconnect the power plug to/from the electrical outlet.
- When disconnecting the plug, pull it out by holding the plug and not by the cord.
- Do not connect or disconnect the plug from the electrical outlet with wet hands. It may result in electrical shock or machine trouble.
- To avoid electrical shock, the power plug and cord must always be dry. Never wet the power plug and cord or touch them with wet hands.
- Do not place the power cord under any objects including the chair as it may damage the cord and result in fire or electrical shock.
- Do not let the mains power cord hang down from the chair. Must be kept clear of all moving parts.
- Before using the chair, check the power cord or plug is not damaged. Do not operate the chair if the power cord or plug is damaged. Do not try to change the damaged plug or cord by yourself. Always contact a qualified technician or the After Sales Service for inspection and repair.
- Do not operate the remote control with wet hands or with sharp or hard objects.
- Do not sit, step or place heavy objects on the remote control.
- Do not operate the remote control with excessive force.
- Do not operate the chair with damaged or broken remote control. It may result in electrical shock. The remote control must be replaced by a new one and must be done by a qualified technician.
- Do not sit or lie on your side or stomach during operation of the chair. You must always sit or lie upright and on your back.
- Do not put your fingers or any object between the seat and footrest.
- Before cleaning or repairing the chair, make sure the main switch is turned off and the power plug disconnected from the electrical outlet.
- Do not use this chair in humid or dusty environment, as it may cause electrical shock or machine trouble. The chair must be in a dry room.
- Do not use this chair in room temperature higher than 40°C.
- Do not wash the chair with water .
- Do not use the chair if the upholstery or any part of the chair is damaged.
- The chair can bear a maximum weight of 150 kg (standard) or 220kg (HD). Person(s) above this weight should not sit on this chair as it may damage the chair.
- Do not remove the upholstery or the back cover from the chair. Touching the inside mechanisms of the chair may result in electric shock.
- The manufacturer of this chair, their agents and distributors are not liable for any damages caused to person(s), object(s), animal(s) or property(s) resulting from improper and misuse of the chair.
- Keep this instruction book for future reference.

When to stop using the chair:


- If water is spilt on the chair. Continuous usage may result in electric shocks.
- If the upholstery or any part of the chair is torn or damaged.
- If the chair has a malfunction during operation.
- If there is a power failure. Injury may occur if the power is restored unexpectedly.
- If you suffer from abnormal or severe pain during the operation of the chair.
- In any of the above events switch OFF the power immediately and disconnect the power cord from the power outlet.

Further notes:

- Check for tears in the upholstery cloth regularly.
- Check all wires, plug and the remote control regularly for any damage.
- Check all wires are free from obstruction and will not be crushed when lifting or lowering the chair.
- Power transformers are all independently tested and tagged in New Zealand, to New Zealand standards.

Assembly Instructions

Unpackaging the Lift Chair

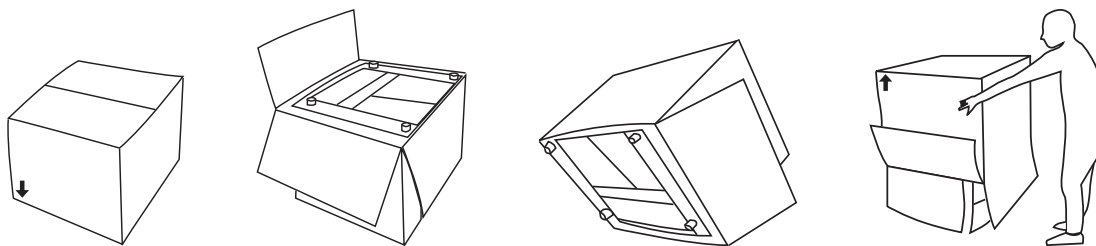


Cut open the top of the box, then carefully flip the box upside down and lift box up to remove.

- !** **IMPORTANT**
Do not attempt to lift the Lift Chair out of the box. Lifting the chair may cause injury due to its weight, or may damage the chair itself.
- !** **IMPORTANT**
Never attempt to pick up the Lift Chair (when in pieces or fully assembled) by holding on to the chair cushions or arms. This may stretch or tear the material of the chair.

Assembling the Chair

The Lift Chair is packed partially assembled, with separate backrest and chair base parts.

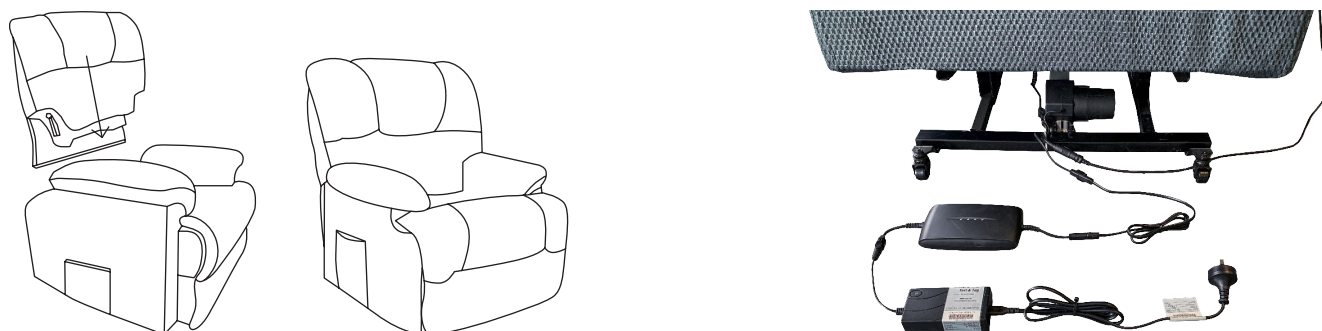

1. Slide the backrest down so that the sleeve engages with the bracket on the chair base. Ensure that these lock into position.
2. Connect all cables to the backrest as shown above.

- !** **IMPORTANT**
Ensure that there are no obstructions between the chair base and footrest when using the lift function.
- !** **IMPORTANT**
Before using the chair, ensure all cords are free of any moving parts.
- !** **IMPORTANT**
Always place the chair on a flat and smooth floor. **Do not** place the chair on an uneven surface.

User Instructions


Positioning the Chair

Ensure that the chair is positioned as described to allow safe reclining and lifting movements.



Remote Control

The remote control can be set up for left or right-handed use.

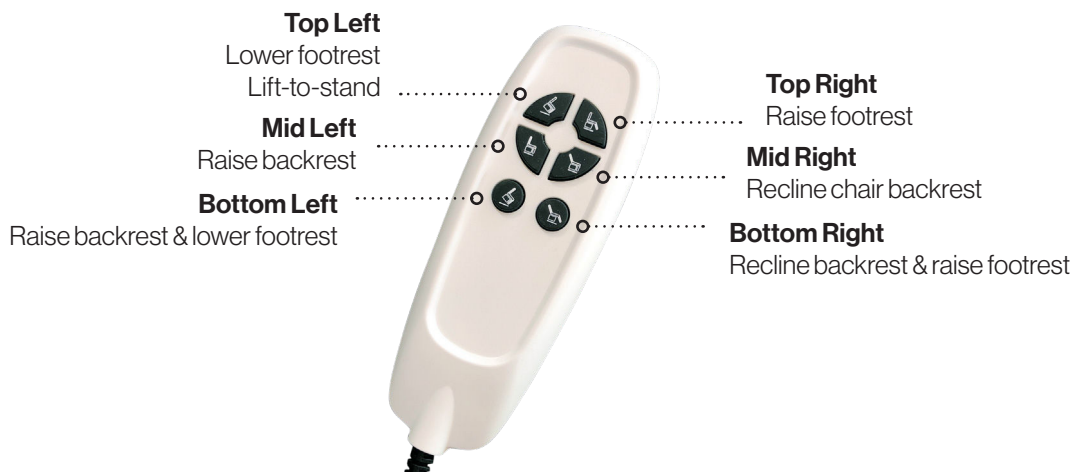

To sit in the chair:

1. While standing in front of the chair, press and hold the Bottom Left button to raise the chair to a suitable height for you to sit down.
2. Press and hold the Bottom Right button to fully lower the chair. Stop pressing the button once you reach a comfortable position - the chair will continue to transition from assisted standing, to sitting, to fully reclined.

To get out of the chair:

1. While sitting in the chair, press and hold the Bottom Left button to raise the chair to a suitable height for you to stand easily.

IMPORTANT

For safety purposes, the chair should be left in the lowered sitting position (footrest lowered, backrest upright) when not in use. This will prevent children, pets or obstructions from becoming trapped underneath the chair.

To elevate the footrest:

1. Press and hold the Top Right button to raise the footrest to a comfortable position.

To lower the footrest:

1. Press and hold the Top Left button to lower the footrest to a comfortable position.

To recline the backrest:

1. Press and hold the Middle Right button until the desired position is reached.

To raise the footrest:

1. Press and hold the Middle Left button until the desired position is reached.

Repairs & Maintenance

As a family-owned and operated New Zealand company, we are committed to delivering quality products that improve daily living with reliability and care. For your safety and the best performance and longevity of this product, please follow the instructions below when your lift chair requires repairs or maintenance.

The symbols below are used throughout this owner's manual to identify warnings and important information. It is very important for you to read and understand them completely.



WARNING

Indicates a hazardous situation that could lead to serious injury or death if it is not avoided.



IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.



CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

Product Repairs

If your chair requires repairs, please contact the After Sales technician to arrange for a professional to carry out the repairs/servicing required. Before any repairs are carried out, always ensure that the power is turned off and the power plug is removed from the electrical outlet.

- The chair must be serviced/repaired by a professional.
- Do not attempt to repair any parts of the chair by yourself.
- Do not change the power plug by yourself.
- Do not disassemble the chair or remove any inside mechanisms by yourself.

Product Care & Maintenance

Before cleaning your chair, always ensure that the power is turned off and the power plug is removed from the electrical outlet.

- The upholstery of the chair must be cleaned with a dry cloth only.
- Do not use any cleaning agents, thinners, or bleaching agents to clean the upholstery.
- Do not wash the upholstery.
- Ironing of the upholstery is strictly prohibited.
- The handles and the rest of the chair must be cleaned with a dry cloth. Never use water to wash/clean any part of the chair.



CAUTION

Do not remove the upholstery for cleaning. Touching the inside of the mechanism of the chair may result in electrical shock.



IMPORTANT

The remote control **must only** be cleaned with a dry cloth. Never use water or a wet cloth, as this may cause damage to the electrical components inside the remote control.

Product Storage

Before storing your chair, ensure that the power is turned off and the power plug is removed from the electrical outlet.

- Ensure that the chair is stored in a clean state. Wipe off any dust and dirt from the chair surface before storing.
- Do not store the chair in high-humidity environments.
- If the chair is not in use for an extended period of time, cover it with a clean sheet to protect it from dust.
- Do not store the chair in direct sunlight or high-temperature environments as the fabric colour may fade.



Goldfern*Mobility*[™]

208 John F Kennedy Drive
Milton, Palmerston North 4414
New Zealand

0800 100 971
hello@goldfernmobility.co.nz
www.goldfernmobility.co.nz