



Premium Rollator Range

8350RIT, 8350RITN185,
8350RIT185, 8350RIT185-HD

Instruction Manual

Read this manual carefully before using this product

Welcome!

Thank you for your purchase of our rollator. The robust design and build of the rollator ensures many years of trouble-free mobility.

Please find the following instructions for use and product specifications. Please read this owner's manual before using your rollator. Improper use of the rollator could result in harm or injury. Correct use improves your mobility and quality of life.

Contents

Welcome!	01
Safety Notice & Caution	02
Instructions & Assembly	03
User Instructions	04
Measurements	05
Maintenance & Other	06

Safety Notice & Caution

- Consult your doctor, therapist or other qualified professional to determine if this rollator will suit your particular needs.
- For proper positioning and use of your Rollator, imagine a line drawn across the back of the Rear Wheels. Part of one foot should always be slightly ahead of that line as you walk.
- This rollator is for individual use only.
- Regularly check the brakes for proper operation. Pulling up on the brake handle will slow the rollator. Pushing down on the brake handle will lock the wheels. Pull up on the brake handle to release the brake lock.
- **DO NOT** use as a wheelchair or transport chair.
- Rollator must be fully opened and in a locked position when being used as a walking aid.
- Wear proper shoes while using this rollator.
- **DO NOT** sit on the rollator and push yourself around using your feet to propel.
- **DO NOT** stand on seat.
- Before sitting on seat, be sure the rollator is fully opened, and the brake is in the locked position.
- **DO NOT** use on stairs or escalators.
- **DO NOT** sit on the rollator if the ground is sloped or uneven.
- Exercise caution when walking on sloped surfaces.
- **DO NOT** use if the rollator is not functioning properly.
- Maximum user weight is 180kg for 8350 RIT 185-HD and 127kg for all other Rollators.
- When storing/transporting your rollator do not place anything on the rollator, as damage may occur.
- Make sure your fingers are clear of the folding or pinch point, whenever the product is folded or unfolded.
- Surface temperature can increase if left in direct sunlight.

These rollators are designed for **indoor & outdoor use**:

- 8350RIT185
- 8350RIT185-HD
- 8350RITN185

These rollators are designed for **indoor use only**:

- 8350RIT

Safe places to hold the rollator while folding or when folded are as follows:

- Handbars
- Lower frame, away from hinging or folding parts

NEVER hold hinging parts while folding, as they can trap fingers and cause injury.



Safety Guidelines

The symbols below are used throughout this owner's manual to identify warnings and important information. It is very important for you to read them and understand them completely.



WARNING! Indicates a hazardous situation that could lead to serious injury or death if it is not avoided.



IMPORTANT! Indicates a hazardous situation that could result in damage to property or rollator if it is not avoided.




CAUTION! Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

Instructions & Assembly


The Premium Rollator comes almost fully assembled, you only need to attach the handles to the rollator. The rollator also comes with:

- Bag
- Food & drink tray (optional)
- Cane holder (optional)

Attaching the Handles

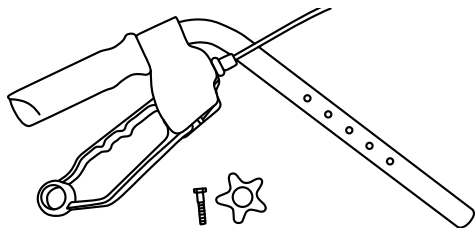

- 1) Gather 2x handles, 2x metal hex screws and 2x plastic screw caps.

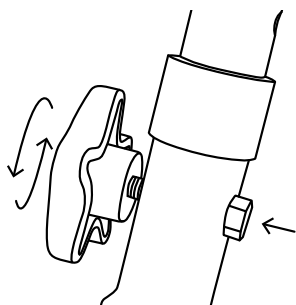

- 2) Insert one handle into the corresponding metal tube on the rollator. Push the handle until the handle is at the desired height, making sure that an inner and outer adjustment hole are aligned.

Insert a hex screw through the two holes, then twist a plastic screw cap on clockwise to hold the handle in place.

Please note: The hex screw will fit comfortably into a hexagonal shape drill hole, and the plastic screw cap will be placed on the outside of the frame.

- 3) Repeat with the second handlebar, screw and plastic screw cap.




CAUTION!

DO NOT use the rollator until you have ensured screws are fully inserted and plastic caps are tightly secured, to prevent handle twisting or loosening.

Folding the Rollator



1. To fold down the rollator, first unclip the clip on the hinge of the rollator.

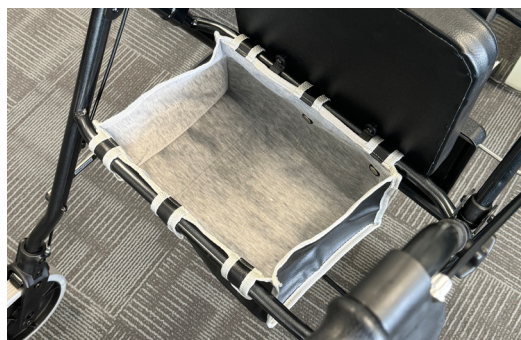

2. Flip back the rollator seat to reveal the storage bag. Ensure that there are no items in the bag before moving to the next step.




3. Grab the fabric at the bottom of the bag, as well as the bar underneath it (indicated by the dashed line). Pull upwards until the rollator cannot fold any more. The rollator is now ready to be transported.



WARNING! DO NOT hold any hinging or moving parts while folding or moving the rollator. Visit the Safety Guidelines page for instruction on where is safe to hold the rollator.

User Instructions

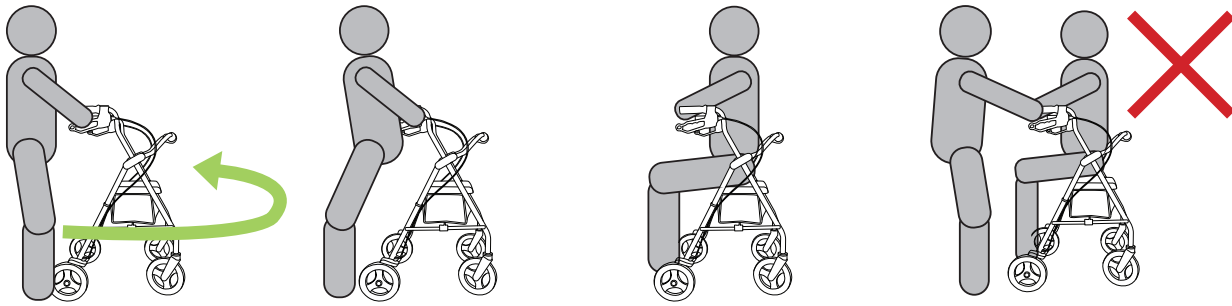

DO NOT support your weight on the handle when attempting to sit. Use the handles to stabilize your body as you carefully turn around. Keep your hands on the handles and slowly lower your weight onto the seat.

Before sitting down on the rollator ensure the brakes are in the parking position and make sure the rollator is on an even surface and will not slide or topple over. Sit up straight and never lean backwards.

DO NOT use as a wheelchair or transport chair.



WARNING!

NEVER move rollator while seated and never sit more than one person on the seat.



WARNING!


NEVER sit on the rollator seat without making sure the back strap is fixed correctly.




WARNING!

ALWAYS apply parking brakes before sitting and only sit on the seat when you are located on a level and firm surface.

Brake Usage

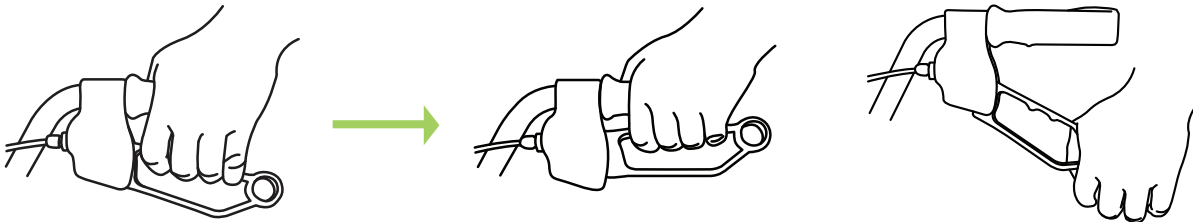

Pull up on brake handle to slow down or to release the parking brake. Push down on brake handle until it clicks to lock the brake into park.

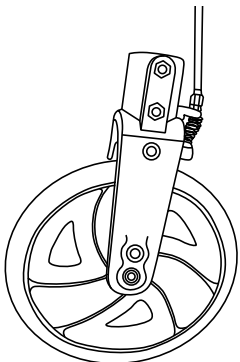

To adjust brake tension, loosen the locking nut, and turn the adjustment screw clockwise/counterclockwise.

Measurements

8350RIT (Premium Rollator 6")

	When open	When folded
Overall width	590mm	590mm
Overall length	642mm	240mm
Overall weight	8.2kg	8.2kg
Handle height	820-907mm	820-907mm

8350RIT185 (Premium Rollator 8")

	When open	When folded
Overall width	585mm	585mm
Overall length	688mm	255mm
Overall weight	8.8kg	8.8kg
Handle height	858-945mm	858-945mm

8350RITN185 (Premium Rollator Narrow 8")

	When open	When folded
Overall width	550mm	550mm
Overall length	670mm	260mm
Overall weight	8.7kg	8.7kg
Handle height	800-890mm	800-890mm

8350RIT185HD (Premium Rollator Heavy Duty 8")

	When open	When folded
Overall width	635mm	635mm
Overall length	690mm	240mm
Overall weight	10.8kg	10.8kg
Handle height	855-942mm	855-942mm

Maintenance & Other

Regular Checking

Brakes:

Examine daily before each use and make sure brakes are working correctly. Brakes must be in the parked/locked position before sitting on the rollator.

Tyres & Wheels:

Examine tyres periodically for wear and make sure they move freely.

Seat:

Examine and check regularly for wear.

Bolts:

Regularly check all frame bolts are fully tightened. Ensure that all bolts are securely fastened before using this rollator.

Cleaning

Your rollator can be cleaned with warm soapy water.

DO NOT use oil or other chemical solutions to clean your rollator.

DO NOT use scrub detergents, steam, or high-pressure cleaners.

Storage

The rollator should be stored in an upright position. It can be stored indoors and outdoors. If outdoors, then it should be stored under a waterproof cover.

Important Information

If you have a visual, reading or cognitive disability, please contact your local physician for help in demonstrating this rollator and its features.

For information about your warranty, please visit www.goldfernmobility.co.nz/warranty-repairs

Attachments

- Vinyl under-seat storage bag
Max load 5kg
- Optional food tray
Max load 10kg
- Optional cane holder



IMPORTANT!

DO NOT use strong corrosive cleaners to clean or polish surface. This could damage the rollator.



IMPORTANT!

Store in a dry location.



CAUTION!

NEVER lift or fold the rollator while holding hinging parts. Safe places to grip the rollator are the handles and lower frame away from hinges or moving parts.



CAUTION!

Be careful when folding and unfolding the rollator to avoid injury by the mechanism.




WARNING!

ALWAYS apply the brakes when in stationary position or when seated on the rollator.



WARNING!


DO NOT hang bags or other items from the handles - this will add stress to the rollator and could also make it unstable.

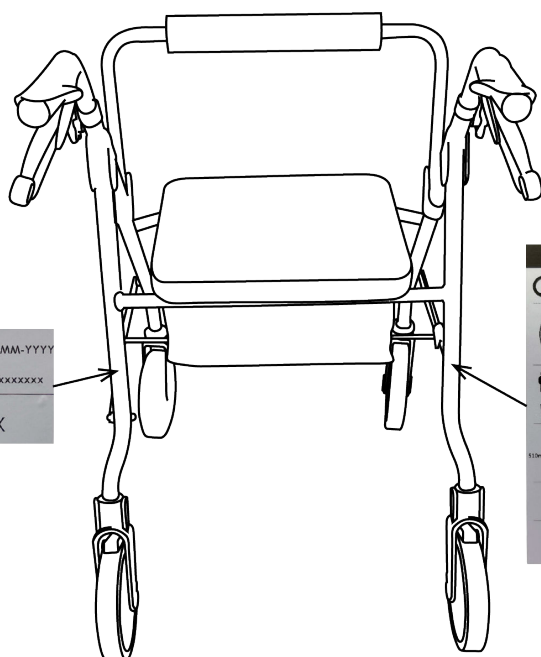


Goldfern*Mobility*[™]




Premium Narrow
Rollator

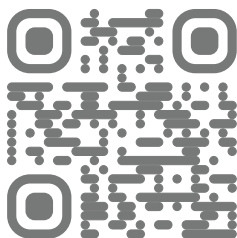

Premium Rollator

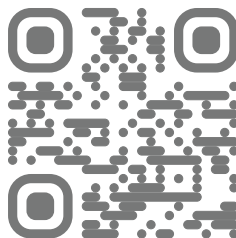

Premium HD
Rollator

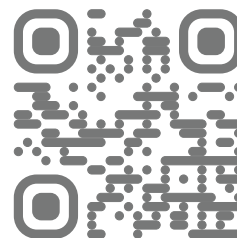

Goldfern Mobility
208 John F Kennedy Drive
Palmerston North
New Zealand

0800 100 971
www.goldfernmobility.co.nz