



Raised Toilet Seat

Assembly Guide & User Manual

Read manual carefully before installation or use.

Assembly & Installation

1. Insert tightening clamps into the base of the Raised Toilet Seat

Turn the Raised Toilet Seat upside down and insert both clamps into the base. Twist clamps only until they are secure enough to remain in place without falling out.



2. Place the Raised Toilet Seat on the upper surface of the toilet bowl


Ensure that the Raised Toilet Seat sits centrally on the toilet, with the tongue section firmly against the front of the toilet bowl.

For easier setup and cleaning, you may remove the existing toilet seat and lid, although this is not required for installation.



3. Tighten clamps until the Raised Toilet Seat sits firmly in place

Twist the screws until the clamps sit tightly against the side of the toilet bowl. The Raised Toilet Seat should not be able to move around on top of the toilet.

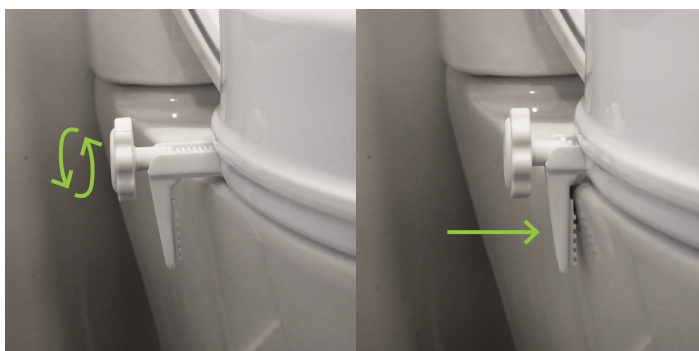

Fitting & Suitability

Goldfern Mobility's Raised Toilet Seats are thoughtfully designed to provide a more comfortable, elevated seating position, enhancing bathroom accessibility and promoting greater independence for users.

As a family-owned and operated New Zealand company, we are committed to delivering quality products that improve daily living with reliability and care.

For your safety and the best performance of this product, the Raised Toilet Seat must always be installed in accordance with the instructions provided in this guide.

While this seat is designed to fit most standard toilet bowls, compatibility cannot be guaranteed across all models. Each toilet must be individually assessed to confirm suitability before use. If correct installation is not possible, do not use the product. If removing the existing toilet seat and lid reveals an uneven or non-flat mounting surface, the Raised Toilet Seat should not be installed.

Product Use & Care

It is strongly recommended that users consult a healthcare professional to determine the appropriate seat height and ensure safe use of this product.

Goldfern Mobility's bathroom and toilet range is intended for indoor use only. The Raised Toilet Seat is fully sealed for hygienic use and easy cleaning.

To clean, simply wipe with a damp cloth and mild detergent. Dry thoroughly with a clean cloth. Avoid the use of harsh, caustic, or abrasive cleaning products, as these may damage the surface and void the warranty. Always ensure soap residue is rinsed away after cleaning.

! **IMPORTANT!**
Do not use strong corrosive cleaners to clean or polish surface. This could damage the Raised Toilet Seat surface.

Important Notice

The user assumes all risk and liability with respect to the correct selection and use of this product. Always consult a qualified health professional to determine the exact adjustment and application of the product for individual needs.

This guide is subject to updates without prior notice.

Visit www.goldfernmobility.co.nz/product/raised-toilet-seat to download the most recent copy of the Raised Toilet Seat Assembly Guide & User Manual.



Goldfern*Mobility*[™]

208 John F Kennedy Drive
Palmerston North
New Zealand

0800 100 971
hello@goldfernmobility.co.nz
www.goldfernmobility.co.nz