



Goldfern*Mobility*™


Freestanding Toilet Surround

Assembly & User Manual



1. To assemble, slide both sides of the Toilet Surround into the ends of the Center Bar (see Diagram 1). Make sure the square holes are all on the same side. You can adjust the width of the Toilet Surround by aligning one of the three holes in each side piece with the holes in the Center Bar.
2. Insert the two provided bolts into the holes, twisting until the square section of the bolt slots into place (see Diagram 2). Place the washer on the back end of the screw, then twist the wing nut on until it is secure.
3. You can now place the Toilet Surround over the toilet. Make sure that the Center Bar is at the front of the toilet, as pictured on the reverse.

Dia 1

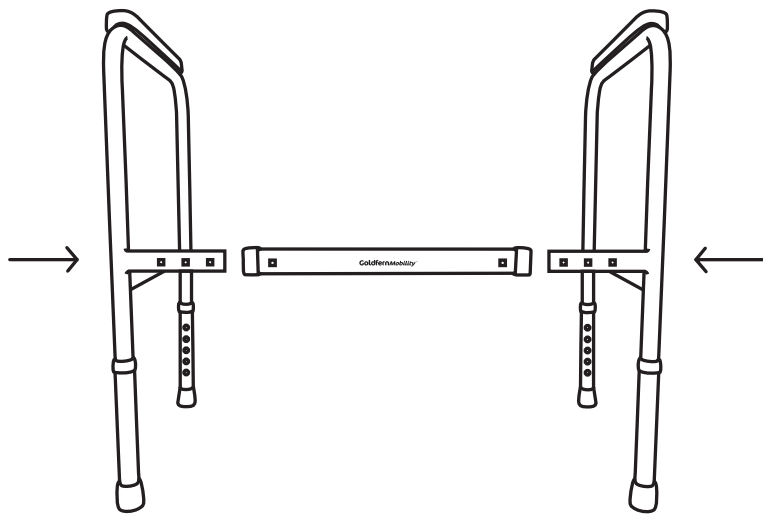

Dia 2

