



Unisex Non-Spill Urinal Bottle

Assembly Guide

Read manual carefully before installation or use.


Assembly Instructions

Adding the Male Attachment:

1. Insert the nozzle and inner tube, tightening enough to hold the inner tube firmly in place.

Firm tightening is adequate; over-tightening the nozzle with excessive force may split the plastic ring.

The raised line on the nozzle should face straight upward and align with the plastic bottle seam when attached.

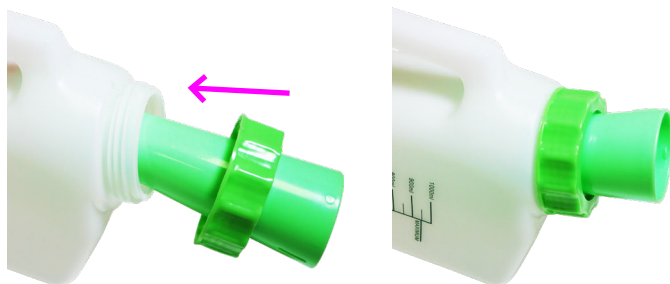

Adding the Female Attachment:

1. Insert the smaller end of the Female Attachment into the green inner tube/Male Attachment.
2. Push and turn clockwise. Turn until the plastic seam on the attachment aligns with the raised green line on the inner tube/Male Attachment.

Not sure? View our Urinal Bottle assembly video at www.goldfernmobility.co.nz/product/urinal-bottle/



Emptying the Urinal Bottle:

1. The Non-Spill Level and Maximum Capacity are marked on the side of the bottle.
2. When you are ready to empty the bottle, simply twist off the green bottle cap and pour out.

IMPORTANT!

- ! Do not attempt to tip the bottle and empty from the front. The non-spill design of the bottle prevents liquid from exiting in this direction.

IMPORTANT!

- ! Take care when re-capping the bottle so that cross-threading does not occur.



Product Suitability

As a family-owned and operated New Zealand company, we are committed to delivering quality products that improve daily living with reliability and care.

For your safety and the best performance of this product, the Unisex Non-Spill Urinal Bottle must always be assembled in accordance with the instructions provided in this guide.

Product Care & Cleaning

It is strongly recommended that users consult a healthcare professional to ensure safe use of this product.

To clean, rinse thoroughly with warm tapwater and mild detergent or multi-purpose cleaner. Dry thoroughly with a clean cloth. Avoid the use of harsh, caustic, or abrasive cleaning products, as these may damage the surface and void the warranty. Always ensure all residue is rinsed away after cleaning.

! IMPORTANT!

- Do not use strong corrosive cleaners to clean or polish surface. This could damage the Urinal Bottle surface.

! IMPORTANT!

- Avoid cleaning with steam, boiling water, or other high-temperature methods, as these can damage or melt the internal non-spill bag. This may lead to leaks during future use.*

Important Notice

The user assumes all risk and liability with respect to the correct selection and use of this product. Always consult a qualified health professional to determine the exact adjustment and application of the product for individual needs.

*Replacement Non-Spill Bags are carried by the distributor if the original becomes damaged.

This guide is subject to updates without prior notice.

Visit www.goldfernmobility.co.nz/product/product/urinal-bottle/ to download the most recent copy of the Urinal Bottle Assembly Guide & User Manual.

Our Assembly Insutructions Video is available at www.goldfernmobility.co.nz/product/urinal-bottle/



Goldfern*Mobility*[™]

208 John F Kennedy Drive
Palmerston North
New Zealand

0800 100 971
hello@goldfernmobility.co.nz
www.goldfernmobility.co.nz